




Osteotome Kit

Achieve greater primary stability in soft bone

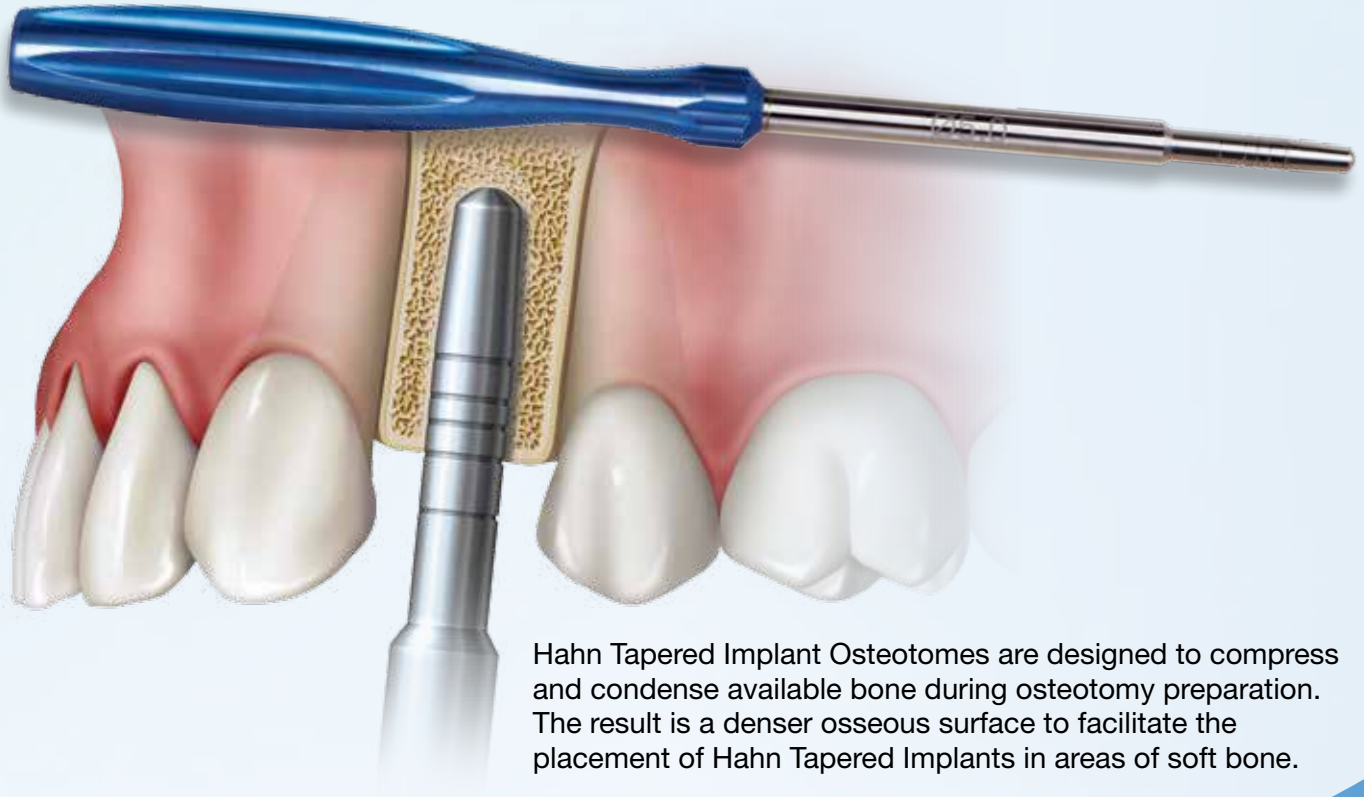

Hahn Tapered Implant Osteotomes are designed to compress and condense available bone during osteotomy preparation. The result is a denser osseous surface to facilitate the placement of Hahn Tapered Implants in areas of soft bone.

~~\$795~~ **Free**

with the purchase of 10 Hahn™ Tapered Implants for \$1,600 (\$160 each)

33% savings off total package value of \$2,395




Hahn™ Tapered Implant Osteotomes not sold individually.

To Order Call:
888-303-3975
hahnimplant.com

GLIDEWELL DIRECT
CLINICAL AND LABORATORY PRODUCTS

Achieve greater primary stability in soft bone

Hahn Tapered Implant Osteotomes are intended for use in the placement of Hahn Tapered Implants in areas of soft bone. As with any surgical procedure, please observe all necessary protocols and precautions.

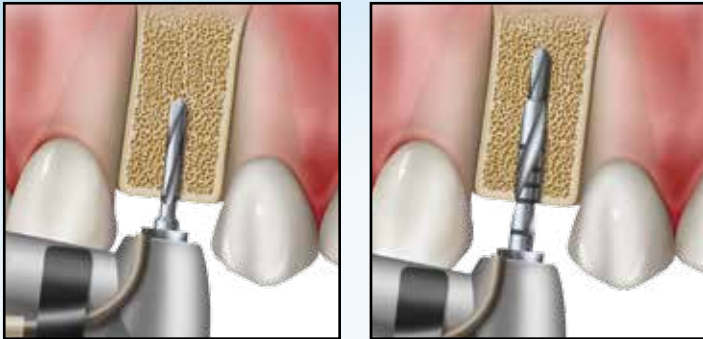

Step 1: Prepare the initial osteotomy with the $\text{Ø}1.5$ mm Twist Drill, followed by preparation with the $\text{Ø}2.4/1.5$ mm Twist Drill to the appropriate depth.



Each Hahn Tapered Implant Osteotome is diameter-specific, to match the diameter of the prescribed Hahn Tapered Implant.



Consult Instructions For Use



$\text{Ø}3.0$ mm



$\text{Ø}3.5$ mm



$\text{Ø}4.3$ mm



$\text{Ø}5.0$ mm

Step 2: Use Osteotomes sequentially to widen the osteotomy to the matching diameter. To avoid overpreparation, widening Osteotome diameters should be used only as needed, and in proper succession.

Cutting-edge design based on 45 years of experience



Official Implant of the

